

*St. Jude Medical[®] vs.
Medtronic Advantage[®]*

A Design and Performance Comparison

PART I:
IMPLANTABILITY &
HEMODYNAMICS

Implantability

St. Jude Medical® Valve

- A large variety of cuff configurations to accommodate surgeons' preferences regardless superb hemodynamics
- Shape: traditional or Flex Cuff
 - Thickness: Standard or Expanded Cuff
 - Material: Double Velour Polyester or PTFE

Medtronic Advantage® Valve


Only 1 cuff option available:

- Shape: traditional
- Thickness: Standard
- Material: Double Velour Polyester

Hemodynamics

Design Objective

- **100% utilization of the native annulus area**
- **Minimal myocardial workload**
- **Minimal turbulence generation**

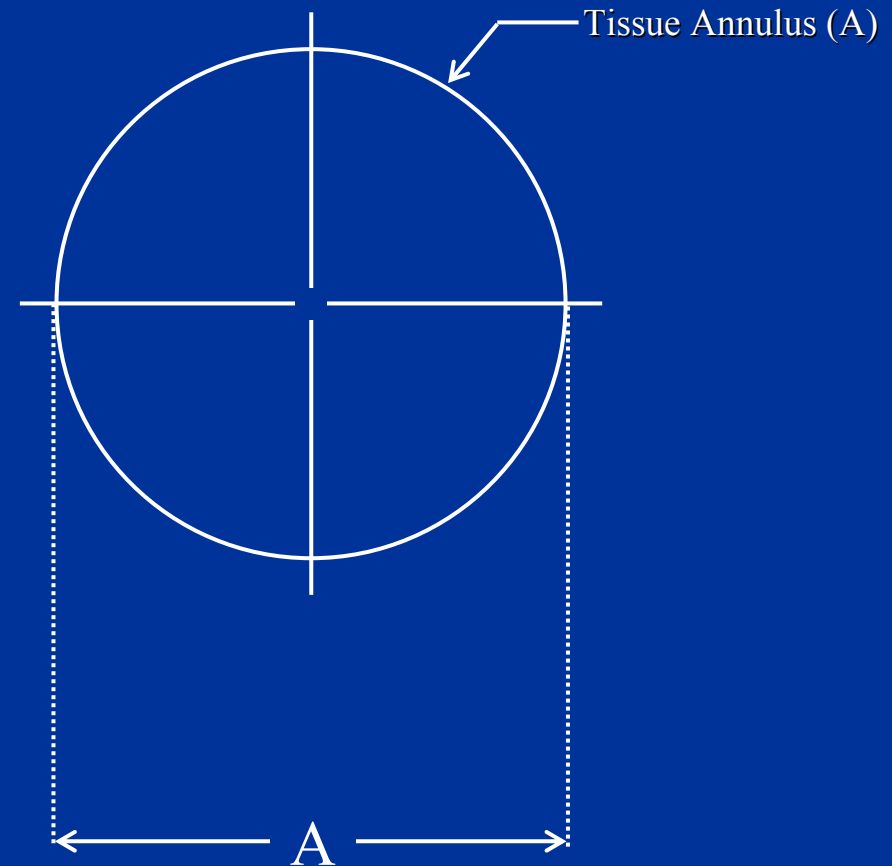


Importance of hemodynamics

Avoidance patient prosthesis mismatch

- Higher transprosthetic gradients
- LVOT obstruction
- Decreased post-operative cardiac index
- Decreased NYHA functional class improvement
- Decreased Quality of Life
- **Higher incidence of late adverse complications** (Pibarot, Patricek)
- **Decreased late survival** (Pibarot, Rao, Rahimtoola)
- **Sudden death** (Kratz, Hachida, Rahimtoola, Bach, Renzulli)

Comparison

Geometric Orifice Areas (GOA)

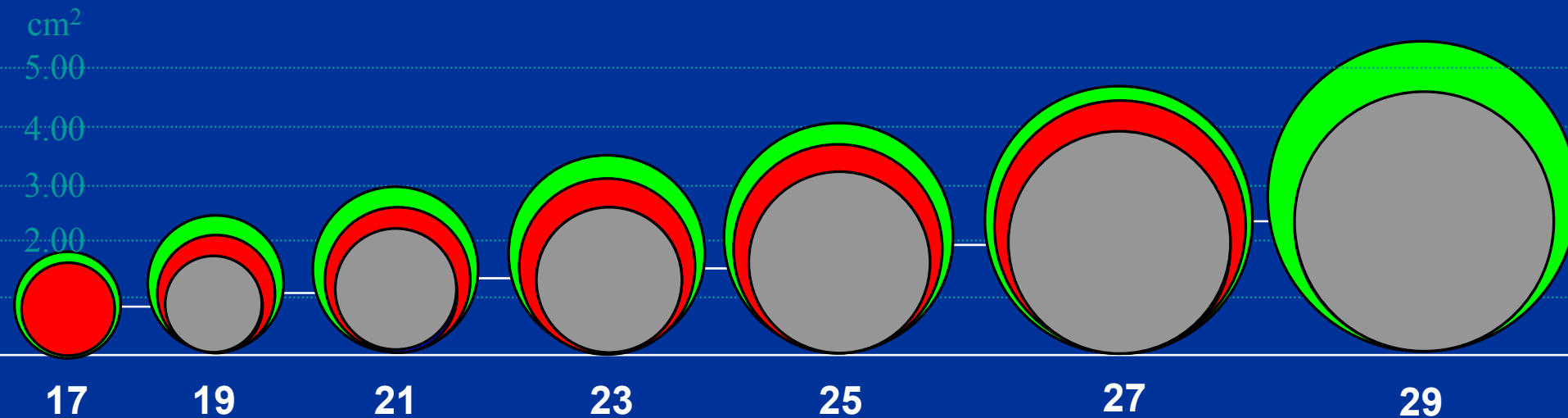

SJM® Standard Valve

SJM® HP Valve

SJM Regent™ Valve


Medtronic Advantage®

SJM Regent : Improvements up to **47%** over the Standard and 17% over the HP

SJM Regent : Improvements up to **41%** over the Medtronic Advantage® Valve

Comparison

Geometric Orifice Areas (GOA)

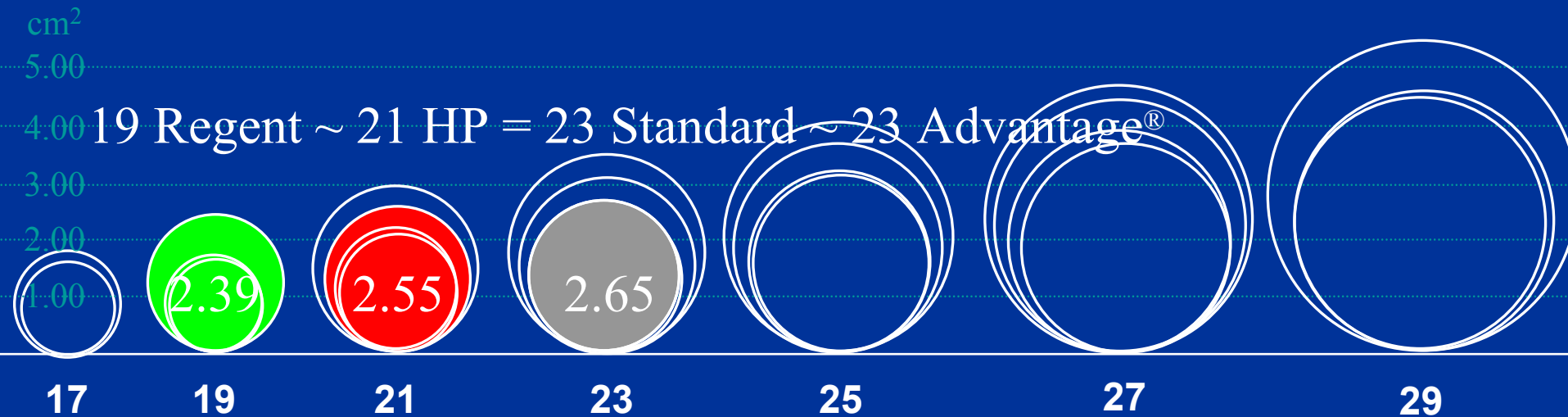

SJM[®] Standard Valve

SJM[®] HP Valve

SJM Regent[™] Valve

Medtronic Advantage[®]

SJM Regent : Improvements up to **47%** over the Standard and 17% over the HP

SJM Regent : Improvements up to **41%** over the Medtronic Advantage[®] Valve

Forward Flow Hemodynamics

St. Jude Medical® Valve



Medtronic Advantage® Valve




Test conditions: compliant to ISO 6540 guideline

Forward Flow Hemodynamics

St. Jude Medical® Valve

Medtronic Advantage® Valve

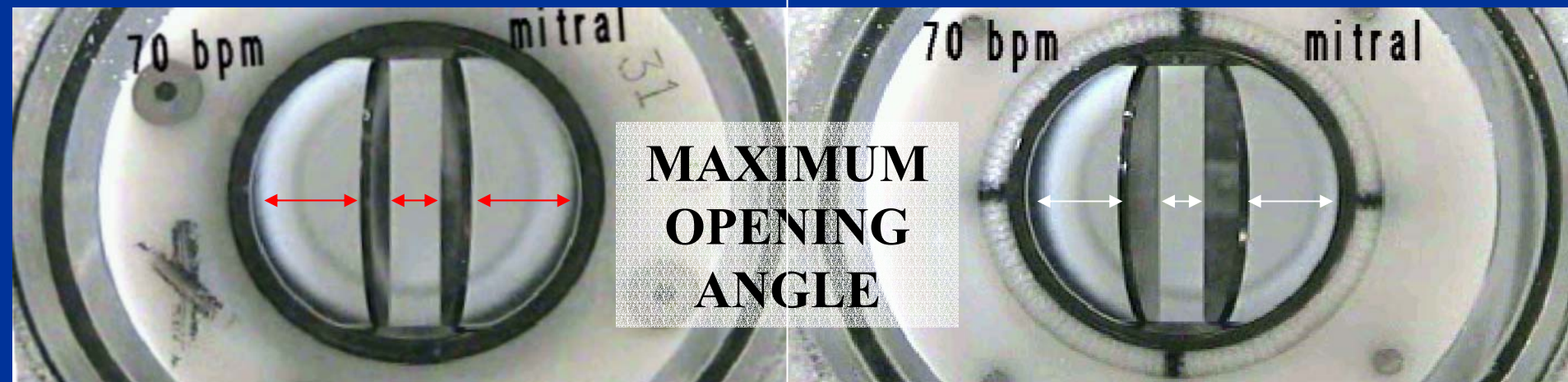


CONCLUSIONS Medtronic Advantage®:

Central vs. lateral **pressure battle on leaflets** leads to:

- Incomplete leaflet opening and flutter resulting in
 - > **SMALLER** effective central opening area
 - > **SMALLER** effective lateral opening area


In-vitro mean gradient and EOA 21mm valves

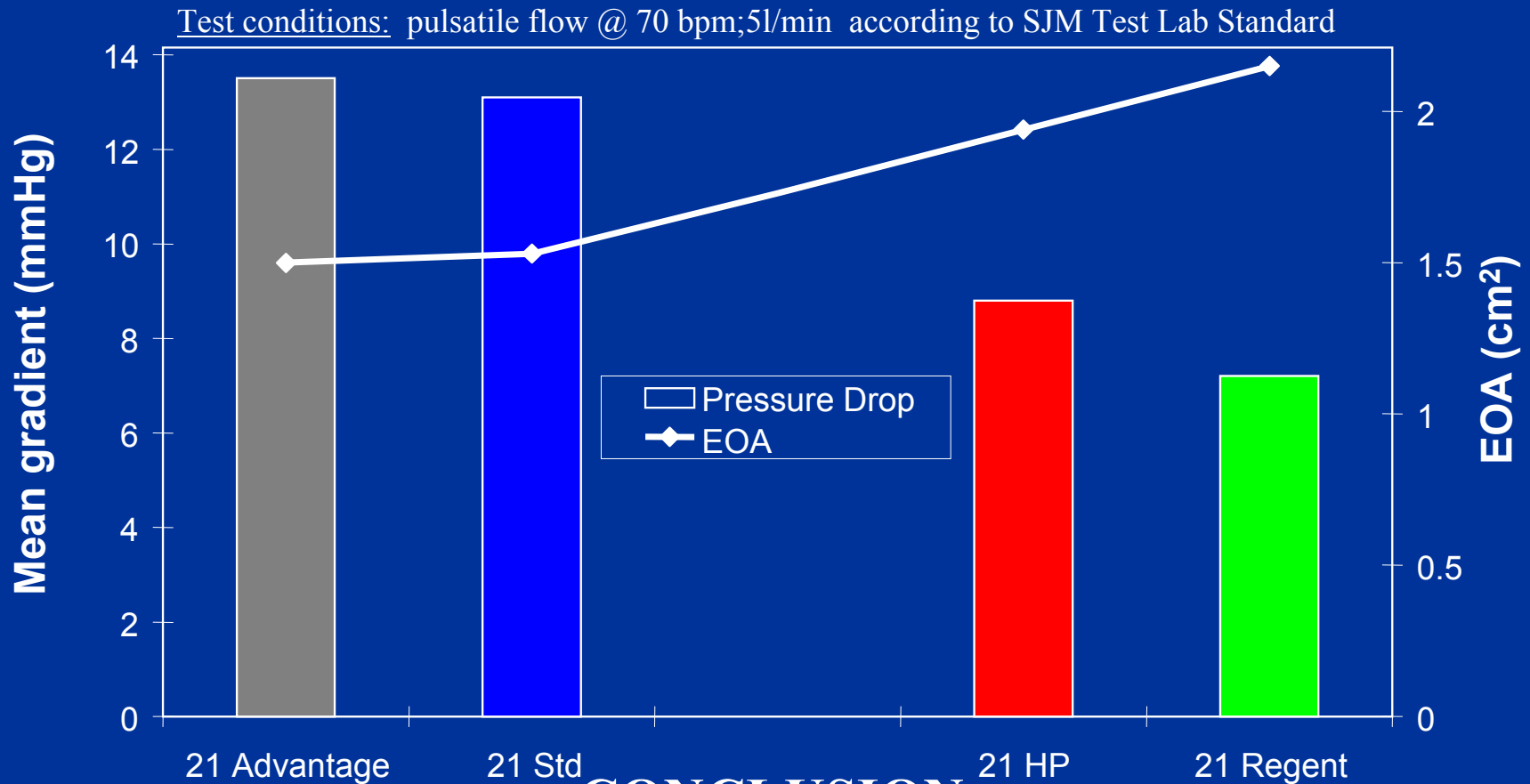

CONCLUSION:

21mm Medtronic Advantage[®] Valve = 21mm SJM[®] Standard Valve

Hemodynamics & Patient-Prosthesis Mismatch

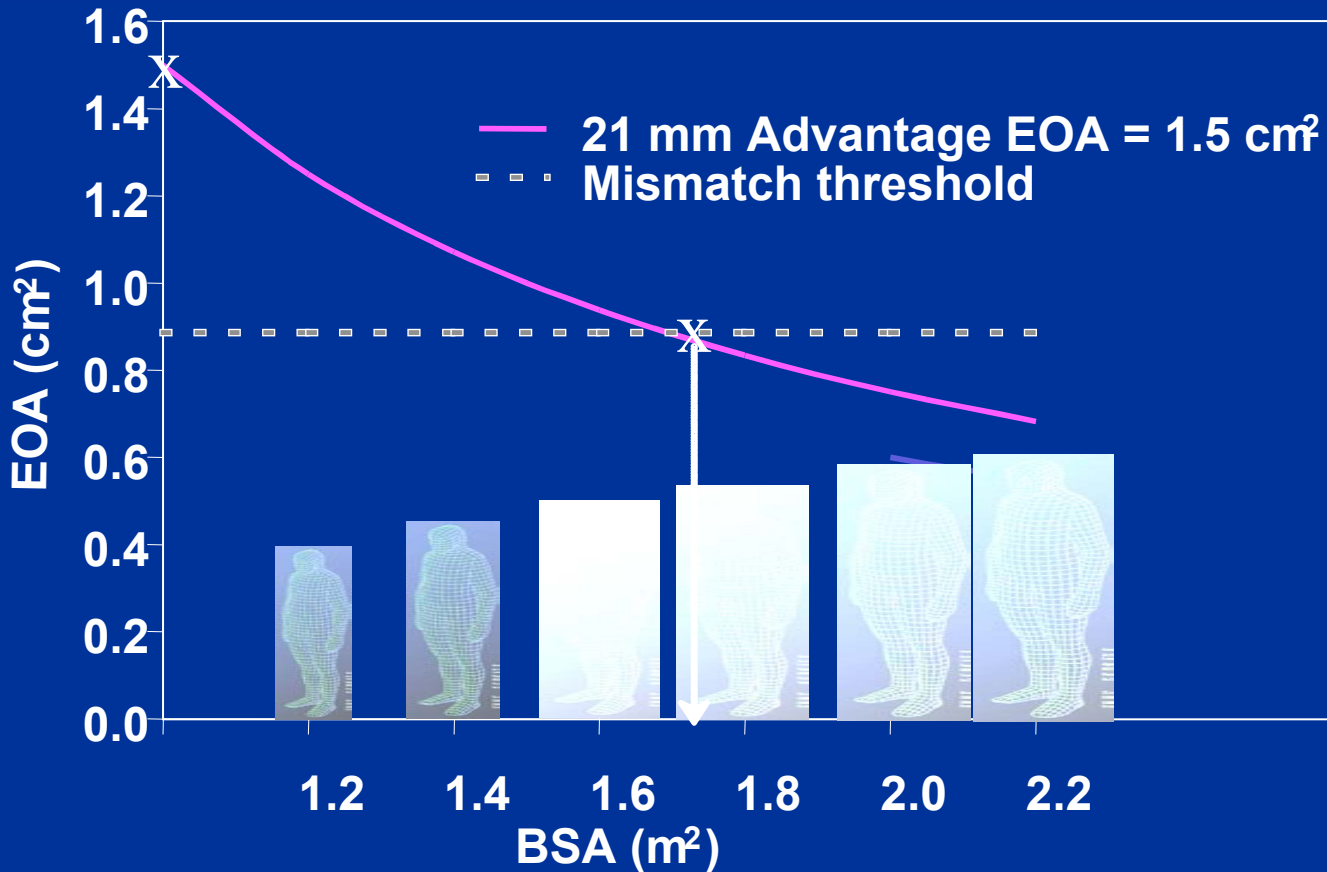

CONCLUSION: 21mm Medtronic Advantage® Valve causes Patient-Prosthesis Mismatch as of BSA > 1.7 m²

*PART II:
THROMBORESISTANCE &
DURABILITY*

Thromboresistance : Manufacturing process differences

St. Jude Medical® Valve

Machining & polishing:

results in a smooth & clean
carbon surface

Medtronic Advantage® Valve

Surface Engineering & polishing:

results in visual bumps on the
carbon surface

**CONCLUSION: Medtronic Advantage® carbon surface
shows visual bumps, which are potential sites for thrombus
formation**

Thromboresistance : Hinge mechanism

St. Jude Medical® Valve

Proven hinge mechanism

28 NOV 2000

Data confirmed *in vivo*

31M-101 S/N 11637

- 19 years implanted
- WEAR : LESS THAN 1 MICRON

Medtronic Advantage® Valve

Copy of St. Jude Medical's hinge mechanism; no proven wear rates



CONCLUSION: Medtronic Advantage® Sure-Flow™ butterfly pivot recesses is a copy St. Jude Medical's hinge recess but without St. Jude Medical Quality Control Label.

Durability : Clinical Evidence

St. Jude Medical[®] Valve

- Proven design: subject of over 1000 clinical papers & > 25 years of clinical history
- Over 1.2 million implants

Medtronic Advantage[®] Valve

- Unproven design: No follow-up data existing
- Less than 1000 implants

CONCLUSION: Medtronic Advantage[®] is a NEW valve with design question marks and no clinical proof

CONCLUSION

What are the advantages of the Advantage[®] valve?

DESIGN CLAIM

FACT

- **Wider central flow area** → “wider” becomes “smaller” due to incomplete leaflet opening
- **SureFlow[™] butterfly pivot system** → unproven copy of St. Jude Medical’s mated sphere pivot system
- **No obstructive pivot guards** → leads to dysfunctional fluttering of valve leaflets
- **Lower velocities** → only lower than caged-ball designs

CONCLUSION


There are no advantages to the Advantage[®] Valve ;
but there are risks....

- Patient-prosthesis mismatch
- Potential for higher TE or thrombus due to unproven “Surface Engineering”
- Turbulence due to leaflet flutter
- Lack of clinical evidence
- Company with history of bileaflet design issues

...so do you want to put your patients at risk ???

Product specifications*

Labeled Size	Valve type	Tissue Annulus Diameter (A) (mm)	Inner Diameter (B) (mm)	Geometric Orifice Area (cm ²)
17	SJM HP	17.0	14.7	1.63
	SJM Regent™	17.0	15.9	1.87
19	SJM Standard	19.0	14.7	1.63
	SJM HP	19.0	16.7	2.06
	SJM Regent™	19.0	17.8	2.39
	MDT Advantage®	19.0	15.1	1.70
21	SJM Standard	21.0	16.7	2.06
	SJM HP	21.0	18.5	2.55
	SJM Regent™	21.0	19.6	2.90
	MDT Advantage®	21.0	16.9	2.13
23	SJM Standard	23.0	18.5	2.55
	SJM HP	23.0	20.4	3.09
	SJM Regent™	23.0	21.4	3.45
	MDT Advantage®	23.0	18.8	2.65
25	SJM Standard	25.0	20.4	3.09
	SJM HP	25.0	22.3	3.67
	SJM Regent™	25.0	23.0	4.02
	MDT Advantage®	25.0	20.6	3.19
27	SJM Standard	27.0	22.3	3.67
	SJM HP	27.0	24.1	4.41
	SJM Regent™	27.0	24.9	4.69
	MDT Advantage®	27.0	22.7	3.90
29	SJM Standard	29.0	24.1	4.41
	SJM HP	29.0	26.0	5.18
	SJM Regent™	29.0	26.8	5.44
	MDT Advantage®	29.0	24.5	4.55
31	SJM Standard	31.0	26.0	5.18
	MDT Advantage®	31.0	24.5	4.55

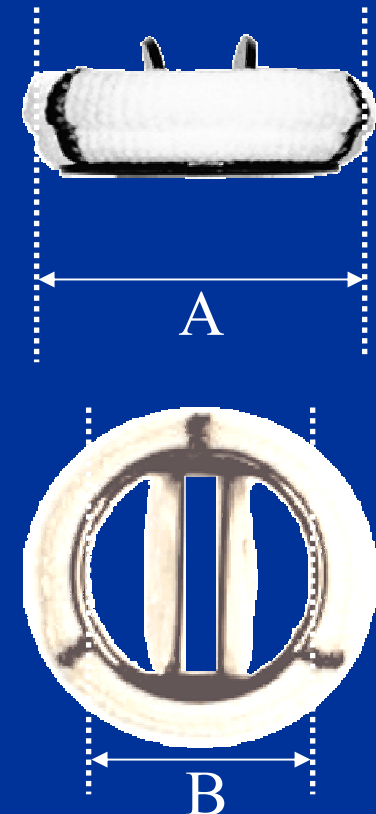

*From manufacturers' data.

Key:

SJM HP = SJM® Masters Series HP
Cuffed Valve

SJM Std. = SJM® Masters Series Standard
Cuffed valve

VISIT OUR WEBSITE AT <http://www.sjm.com>

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

St. Jude Medical prosthetic heart valves are indicated for use as replacement valves in patients with a diseased, damaged, or malfunctioning native or prosthetic valve. Possible side effects for all valvular implants include, but are not limited to: regurgitation, thromboembolic phenomena, resistance to flow, infection, hemolysis, dysrhythmias, and prosthetic dehiscence or failure. Anticoagulation is recommended for patients with mechanical valve implants. Please see the physician's manual for a full description of indications, contraindications, side effects, precautions, warnings and instructions for use.

Corporate Headquarters St. Jude Medical, Inc. One Lillehei Plaza, St. Paul, Minnesota 55117 USA **24-Hour Technical/Professional Consultation** (800) 328-9634 (USA) (651) 483-2000 Fax: (651) 482-8318 **Customer Service** (800) 544-1664 (USA) (651) 490-4410 Fax: (651) 481-7702 **European Headquarters** SJM International, Inc., The Corporate Village, Figueras Building, Avenue Da Vincilaan, 11, 1930 Zaventem, Belgium **Customer Service** Tel: 32-2-774-68-11 Fax: 32-2-772-83-84 **Asian Headquarters** St. Jude Medical Hong Kong Limited, Room 2705-2708, China Merchants Tower, Shun Tak Centre, 168-200 Connaught Road, Central, Hong Kong Tel: (852) 2996-7688 Fax: (852) 2956-0622

Advantage and Sureflow are trademarks of Medtronic, Inc. SJM Regent and The Global Leader in Heart Valve Devices are trademarks of St. Jude Medical, Inc. St. Jude Medical Cardiac Surgery Division. US Patent Nos. 6,007,577; 6,358,278B1; 6,391,053B1; patents and other foreign patents pending. © 2003 St. Jude Medical, Inc. All rights reserved.

ITEM **XXXX**/0303/XM/EN